Do not use if pouch has been damaged or opened.
Niet gebruiken indien de verpakking beschadigd of geopend is.
Nicht verwenden, wenn der Beutel beschädigt oder geöffnet ist.
Må ikke bruges, hvis posen er beskadiget eller har været åbnet.
Ei saa käytää, jos pussi on vaurioitunut tai avattu.
Ikke bruk hvis forpakningen er skadet eller åpnet.
Não utilizar se a embalagem tiver sido danificada ou aberta.
Non utilizzare se la busta è stata aperta o danneggiata.
勿使用已损坏或打开的包装袋。
如果袋子已损坏或打开，请勿使用。
لاستخدام الصناديق إذا كان الصناديق مكسورة أو مفتوحة.
주미나는 손상되었거나 열려있을 경우에는 사용하지 마십시오.
Nepoužívajte, ak je vrecúško poškodené alebo otvorené.
Ne pas utiliser si l'emballage du pansement a été ouvert ou est endommagé.
No utilizar si el estuche está dañado o abierto.

Keep away from sunlight and heat.
Uit de buurt van zonlicht en warmte houden.
Vor Sonnenlicht und Hitze schützen.
Må ikke opbevares i direkte sollys og varme.
Håll borta från solljus och varme.
Säilytävässä suojaudu taa auringonvalolta ja kuumuudelta.
Ikke utsett produktet for sollys og varme.
Manter longe da luz solar e calor.
Tenere lontano dalla luce solare e dal calore.
 Mantener lejos del sol y del calor.
远避阳光和热。
يُنْهَى بِعِدَّةٍ عَنْ أَشْعَرِ النَّهَارِ وَالْمَوَاطِنِ.
帶著它並避免在太陽光下或熱處使用。
Udržívať mimo dosahu priameho slnečného svetla a tepla.
Garder à l'abri des rayons du soleil et de la chaleur.
Mantener fuera del alcance de la luz solar y el calor.

Store ≤ 30°C (86°F).
Bewaren ≤ 30°C (86°F).
Bei ≤ 30 °C (86 °F) lagern.
Opbevaring ≤ 30 °C (86 °F).
Förvara i ≤ 30 °C (86 °F).
Säilytä lämpötila ≤ 30 °C (86 °F).
Lagre ved ≤ 30 °C (86 °F).
Armazenar em ≤ 30 °C (86 °F).
Conservare a ≤ 30°C (86°F).
(86°F) 30°C ≤ の場合、貯蔵温度を 30°C (86°F) 以下に保つこと。
储存温度 ≤ 30°C (86°F).
التخزين 30° درجة سلبيات (86°F).
30°C (86°F) 이하에 보관.
Skladujte do ≤ 30°C (86°F).
Conserver à une température ≤ 30 °C (86 °C).
Conservar a ≤ 30°C (86 °F).

Not made with natural rubber latex.
Niet gemaakt met natuurlijke rubberlatex.
Nicht aus Naturkautschuklatex hergestellt.
Ikke lavet med naturlig gummilatex.
Produkten är inte tillverkat av naturgummilatex.
Ei sisällä luonnonkummitekstia.
Ikke laget av naturlig latex.
Não é feito com látex de borracha natural.
Privo di lattice di gomma naturale.
לטיק טבעי לא מיוצר.
不是采用天然橡胶乳胶制成。
غير مصنوع من لaticex الطبيعي.
천연 고무 라텍스로 만든 제품이 아닙니다.
Vyrobéné bez prirodného latexu.
Ne contiene latex natural.
No están hechos de látex de caucho natural.
Desechables. No reutilizar.

Single use. Do not re-use.
Eenmalig gebruik. Niet hergebruiken.
Zum Einmalgebrauch. Nicht wiederverwenden.
Engangsbrug. Må ikke genbruges.
Für engangsbruk. Ateranvänd inte.
Kertakäyttöinen. Ei saa käytää uudelleen.
Til engangsbruk. Ikke bruk om igjen.
Usa unico. Não reutilizar.
Monouso. Non riutilizzare.
רשות לטלית אחת.
一次性使用。请勿重复使用。
للاستخدام одноразовый.
1회용. 재사용하지 마십시오.
Len na jedno použití. Nepoužívajte opakovane.
À usage unique. Ne pas réutiliser.
Desechables. No reutilizar.

More healing
less pain

FERRIS MFG. CORP.
5133 Northeast Parkway, Fort Worth, TX 76106-1822 U.S.A.
Toll Free U.S.A.: 800-765-9636 • International: + 1 817-900-1301
www.polymem.com • info@polymem.com

© 2016 Ferris Mfg. Corp. • ® or ™ indicate trademark

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EN PolyMem® Non-Adhesive Pad or Roll Dressing

The proven multifunctional dressing.

DESCRIPTION

PolyMem polymeric membrane dressings are made of moisture-loving polyurethane matrix that contains (1) safe wound cleanser, (2) glycerin moisturizer, and (3) superabsorbents. Dressings have a thin polyurethane backing which breathes and helps keep out contamination. Dressings may be used as a primary, secondary, or combined primary and secondary dressing.

PolyMem MAX™ dressings have the same polymeric membrane as PolyMem dressings, but Max is thicker and has higher MVTR to handle more fluid.

Drug-free PolyMem dressings help to:

- effectively manage and heal wounds
- absorb fluid and provide a moist healing environment
- minimize and relieve persistent and procedural wound pain
- reduce edema, bruising, and the spread of inflammation into surrounding undamaged tissues

INDICATIONS

Over-the-counter, for management of minor wounds, such as:

- Abrasions • Lacerations • Cuts • Burns

Under the direction of a healthcare professional, for the management of serious wounds, such as most partial and full thickness wounds, including:

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EN DIRECTIONS**Initial Application**

1. Prepare the wound according to protocol or as directed by a physician or other ordering clinician.
2. Select a dressing with the membrane pad 1/4" – 2" (0.6 – 5 cm) larger than the wound. The dressing should also cover any inflamed or damaged area surrounding the wound. Dressings may be cut.
3. For dry wounds, moisten dressing slightly prior to application.
4. Apply film side out (so printing is visible).
5. Secure dressings, such as with tape, netting, or gauze.
6. Outline the wound on top of dressing to help determine when to change.

Use

1. A dramatic increase in wound fluid may be observed during the first few days due to the fluid attraction. This is not uncommon and indicates that the dressing is working.
2. Keep the dressing dry and in place when bathing. Change the dressing if it gets wet.

Dressing Change

1. For an exuding wound, fluid will become visible through the top. Change before fluid reaches the wound margin, or when good practice dictates, or after no more than 7 days. Change immediately if fluid reaches the edge of the membrane pad.
2. More frequent changes may be needed due to a compromised immune system, diabetes, infection, hypergranulation, macerated tissue, or when desiring to facilitate the removal of non-viable tissue.

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3. Gently remove the dressing. The PolyMem pad will generally not adhere to the wound bed, usually ensuring pain-free dressing changes. Inspect the dressing and wound to assure removal of all material.

4. If coagulated blood or dried exudate has caused the dressing to adhere to skin or hair, use a small amount of saline or water to slowly gently soften and loosen. Clean the intact skin around the wound.

5. Manage dressing changes according to protocol or as directed by physician or other ordering clinician. In most cases, when using PolyMem, there is no need to disturb or cleanse a wound during changes unless the wound is infected or contaminated. PolyMem is designed to continuously clean the wound and does not leave residue that needs to be removed. Excessive cleaning may injure regenerating tissue and delay wound healing.

6. Apply a new dressing.

Observations during use or change

- During normal healing, the wound may appear to become larger during the first few dressing changes because the body is naturally removing non-viable tissue. If the wound continues to become larger, consult an expert.
- PolyMem dressings help support new blood vessel formation so it is not uncommon to see blood-stained wound fluid and dressings during changes.
- Be alert for signs and symptoms, such as, increased redness, inflammation, pain, warmth, odor, whitening, fragility, excessive tissue formation, or lack of healing. Consult a professional for problem situations, such as, infection, irritation, sensitivity, allergy, maceration, or hypergranulation.