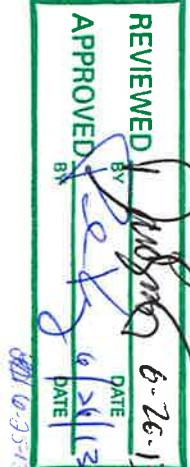
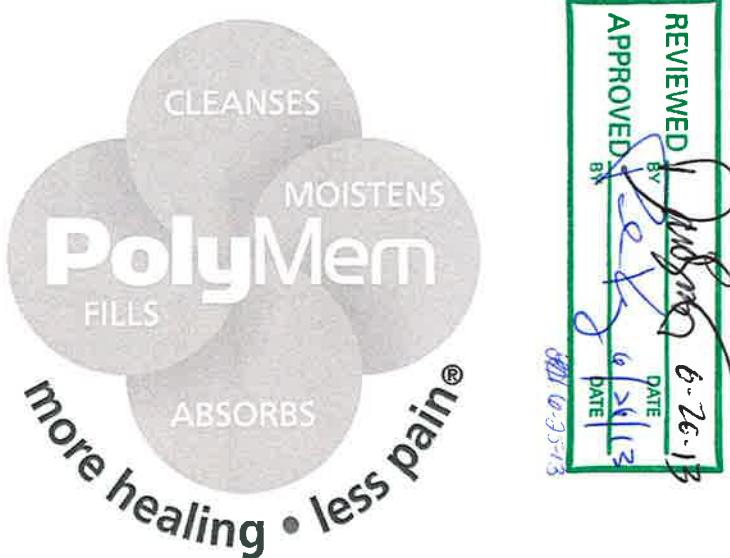
Do Not Reuse. Reuse may cause contamination. No use las almohadillas más de una vez. Su reutilización puede producir contaminación. Ne pas réutiliser. Sa réutilisation risque de causer une contamination. Nicht wieder verwenden. Wieder Verwendung kann Kontamination verursachen. Niet opnieuw gebruiken. Hergebruik kan verontreiniging veroorzaken. Må ikke genbruges. Genbrug kan forstørre kontamination. Får ej återanvändas. Återanvändning kan orsaka kontamination. Ei sei käytätkää uudelleen. Uudelleenkäyttö voi aiheuttaa kontaminatiota. Må ikke brukes igjen. Gjenbruk kan forstørre kontamination. Não reutilizar. A reutilização pode provocar contaminação. אלא לשובת ישנות יותר. שימוש חוזר יכול לגרום ל contamination.	Sterile if Pouch is Not Open or Damaged Estéril si la bolsa está intacta Stérile lorsque l'emballage est intact Steril, wenn der Beutel intakt ist Steril indien zak onbeschadigd is Steril, hvis posen er ubrudt Steril då påsen är intakt Sterili, mikäli pussi on koskematon Steril hvis posen er intakt Estéril se a bolsa estiver intacta Steril if pouch is intact	Latex Free Dressing and Packaging Libre de latex Sans latex Latexfrei Latexvrij Latexfrei Latexfri Latexfri Latexfri Isento de latex כטול לטקס

# PolyMem®

- [REF] 4401 #1 Finger/Toe Dressing
- [REF] 4402 #2 Finger/Toe Dressing
- [REF] 4403 #3 Finger/Toe Dressing
- [REF] 4404 #4 Finger/Toe Dressing
- [REF] 4405 #5 Finger/Toe Dressing

- [REF] 1401 #1 Silver Finger/Toe Dressing
- [REF] 1402 #2 Silver Finger/Toe Dressing
- [REF] 1403 #3 Silver Finger/Toe Dressing
- [REF] 1404 #4 Silver Finger/Toe Dressing
- [REF] 1405 #5 Silver Finger/Toe Dressing

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

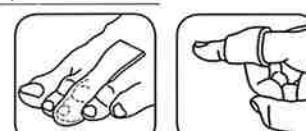
## PolyMem® Finger/Toe Dressing

*The dressing provides a soothing wrap shaped to fit.*

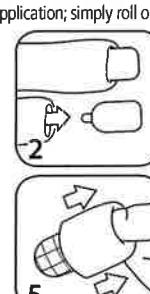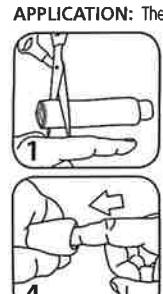**APPLICATION:** The dressing was designed for easy application; simply roll onto the finger.

The grid and words should be on the outside of the dressing.

Maintain comfort. If it feels too tight, remove and apply a larger size. Remove by rolling off, opposite of application.

The dressing may be applied in different ways. For toes the dressing may be cut along the sides creating flaps that can be laid upon the top and bottom of the foot. These can be secured with tape, or the dressing may be cut to form a ring or sleeve over the injured portion of the finger.

PolyMem dressings are made of a moisture-loving polymeric membrane pad that contains (1) a safe wound cleanser, (2) glycerol moisturizer, and (3) absorbents. The absorbent membrane pad is covered by a continuous thin film backing which helps provide a barrier to contamination and has oxygen and moisture vapor permeability. PolyMem may be used as a combined primary and secondary dressing as a secondary dressing. This highly conformable and absorbent dressing draws wound fluid, which is known to contain natural growth factors and nutrients, to the wound site.

PolyMem Silver products incorporate small particles of silver (124 micrograms/cm<sup>2</sup>; ≈ 2,000 parts/million.) The silver protects the dressings from microbial contamination. PolyMem Silver dressings will not stain the skin.

Drug-free PolyMem dressings • help to avoid and relieve persistent and procedural wound pain • help to reduce swelling, bruising, and the spread of inflammation into surrounding undamaged tissues

**INDICATIONS:** For the management of most partial and full thickness wounds, including

- Pressure ulcers      • Acute wounds      • Skin tears      • Dermatologic disorders      • Surgical wounds
- (stages I-IV)                • Donor and graft sites      • Diabetic ulcers      • 1st and 2nd degree burns      • Vascular ulcers

**CONTRAINDICATIONS:** None known

**PRECAUTIONS:** • For external use only • PolyMem dressings are not compatible with oxidizing agents (hydrogen peroxide or hypochlorite solutions) • Do not resterilize • Do not use if package is open or damaged • Single use • Prior to performing imaging procedures, determine if the dressing should be removed • Avoid contact with electrodes or conductive gels • Topical treatments are not recommended in conjunction with PolyMem dressings • Be alert for signs of infection or maceration

All PolyMem dressings may be used when signs of infection are present if proper medical treatment that addresses the cause of the infection has been implemented.

PolyMem Silver dressings are recommended when a wound displays delayed healing, signs of infection, increased bioburden, or the patient has infection risk factors (such as contaminated wound, poor nutrition, diabetes, history of infection, etc.)

### Storage:

- Keep dry and store at room temperature and humidity.
- Expiration Date is 3 years from Date of Manufacture, shown on packaging.

**Dimensions:** Dimensions and measurements on packaging and labeling are approximate.

### DIRECTIONS:

#### Initial Application:

1. Prepare the injured area according to protocol or as directed by a physician or other ordering clinician. Clean the skin of dirt, topical medications and lotions. Residual moisture on the skin is beneficial.
2. Apply the dressing as outlined. A PolyMem dressing should also cover any inflamed or damaged area surrounding the injury. Dressings may be cut to accommodate the injury or body part.
3. Apply the dressing so the film side is out (printing is visible)

#### Helpful Hints:

- Secure dressing with water-proof roll tape to reduce the chance of the inside of the dressing becoming wet.
- Outline the approximate wound margin on the outside of the dressing to help determine when to change.

#### Initial Use:

1. A dramatic increase in wound fluid may be observed during the first few days. This is not uncommon, and indicates that the dressing is working.

2. Keep the dressing dry and in place when bathing or performing hand-hygiene. Change the dressing if it gets wet.

#### Dressing Change:

1. For an exuding wound, fluid will become visible through outside of the PolyMem dressing. Change before fluid, visible through the dressing, approximates the wound margin, when good practice dictates, or after no more than 7 days. Change immediately if fluid fills the membrane pad.
2. More frequent changes may be needed if the patient has a compromised immune system, diabetes, infection at the wound site, or when desiring to speed up the removal of non-viable tissue in the wound.
3. Gently remove the dressing from the wound site. The PolyMem pad will generally not adhere to the wound bed, usually assuring pain-free dressing changes. Inspect the dressing and wound to assure removal of all material.
4. DO NOT DISTURB THE WOUND BED. DO NOT CLEAN THE WOUND BED or flush with saline or water unless the wound is infected or contaminated. PolyMem dressings are designed to continuously clean the wound bed while in place. The dressing does not leave residue that needs to be cleaned from the wound. Additional cleaning may injure regenerating tissue and delay wound healing. If dried exudate has caused the dressing to adhere to intact skin adjacent to the wound, use a small amount of saline or water to gently soften and loosen. Clean the intact skin around the wound while leaving the wound bed undisturbed.
5. Apply a new dressing.